User Manual

Scan QR code

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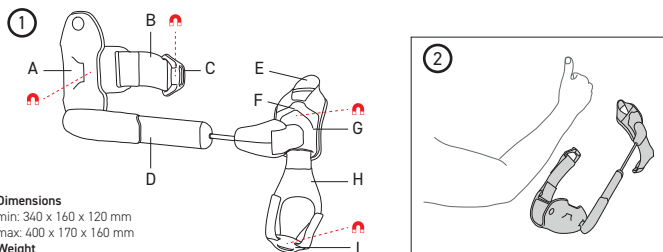
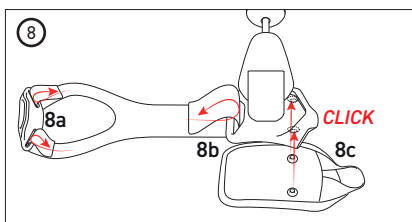
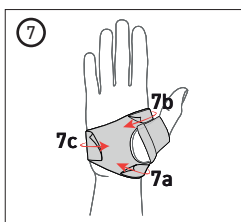
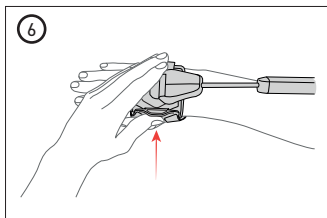
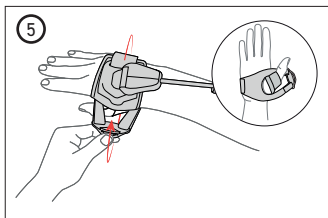
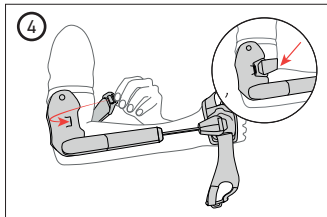
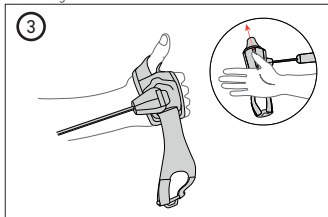
English

Introduction

The STIL Orthosis is an elbow-wrist-hand orthosis for the alleviation of lower arm and wrist tremors. It reduces tremors by means of mechanical damping. The orthosis' performance in suppressing tremor may vary from person to person.

Intended use

The orthosis is intended to be worn on an arm affected by distal action tremors predominantly involving wrist flexion-extension and forearm pronation-supination. It supports the user while performing activities of daily living, such as eating or drinking.

This orthosis is for a single patient - multiple use. Gradually increase the time you wear the orthosis, following the schedule recommended by your healthcare professional. After your skin and muscles have adapted to wearing the device, it is advised to limit use to 2 hours at a time and 5 hours per day.

Indications

The orthosis is intended for users of 18 years of age or older, diagnosed with essential tremor. Suitability of the orthosis for the user must be evaluated by a trained healthcare professional. The orthosis must be installed by a trained healthcare professional.

Contraindications ⚠

This product contains magnets, which can be removed upon request. If the magnets are removed, this contraindication no longer applies. Unless approved by your doctor, do not use the product if you or anyone nearby has a medical device affected by magnets, such as pacemakers, implantable cardioverter-defibrillators (ICDs), deep brain stimulation (DBS) systems, neurostimulators, cerebrospinal fluid shunts, insulin/infusion pumps, aneurysm treatments, stents, heart valve prostheses, ocular implants, metallic gastrointestinal clips, or metallic splinters in the eye.

Side effects

Use of the orthosis may cause muscle fatigue, muscle pain, skin irritation, redness of the skin or discomfort in the hand, elbow, or shoulder. Users with a history of shoulder conditions (e.g., frozen shoulder, rotator cuff injuries) may be more sensitive to shoulder discomfort. Stop using the orthosis and contact your healthcare professional if you experience skin lesions, bruising, cuts, pain or significant discomfort.

Warnings and important notes ⚠

- Consult your physician before discontinuing any tremor-related medication.
- Do not wear the orthosis during sleep.
- Keep all magnets in the closure mechanisms of the orthosis (see Figure 1) at a safe distance of at least 20 cm from implants or medical devices that may be affected by magnetic fields.
- If you experience pain or reduction of product performance, stop using the orthosis and contact your healthcare professional.
- Consult your healthcare professional before using the orthosis if you currently have, or used to have, a shoulder injury (e.g. frozen shoulder, rotator-cuff injuries)
- After venipuncture (e.g. blood draw or intravenous (IV) line insertion), wait at least 2 weeks before using the orthosis on that arm. After an arterial puncture in the arm (e.g., arterial line or catheterization), consult your physician when it's safe to use the orthosis on that arm.
- Use device with caution if you have muscle atrophy. With sustained tingling sensation or numbness in the arm or fingers after after using the orthosis, stop using the device immediately and contact your healthcare professional.
- Consult your healthcare professional before using the orthosis if you have a muscle or tendon injury in the affected arm.
- Consult your healthcare professional before using the orthosis if you have irritated skin, swelling, varicose veins, use corticosteroids (e.g. Prednisone) or have other skin conditions. Inspect the skin under straps and cuffs regularly. If you experience changes in the skin or underlying tissue, stop using the orthosis and consult your healthcare professional.
- Do not wear the orthosis directly on damaged skin (such as open

wounds, skin ulcers, burned areas, infected tissue, cancerous lesions). Consult your physician for guidance - especially for diabetic patients.

- Use with caution if you are taking blood thinners (anticoagulants). The orthosis may increase the risk of bruising due to pressure on the arm. Stop using the device if this occurs.
- Use with caution if you have arthritis in the thumb (CMC joint) or shoulder (glenohumeral joint).
- The orthosis partially limits mobility. Do not perform tasks (such as driving a vehicle, or handling sharp objects) while wearing the orthosis.
- Wash fabrics components before use and at least once every week with disinfectant. Failure to do so may cause skin irritation. It is recommended to replace the fabrics yearly.
- The orthosis is designed for a maximum lifetime of 2 years.
- Keep the orthosis away from dust, dirt, and sand.
- Keep the orthosis dry.
- Do not use the orthosis over watches, bracelets, glucose sensors, or other wearable devices.
- The orthosis may damage clothing.
- Do not wear the orthosis under tight clothing.

Fitting instructions

Before using the orthosis for the first time, practice putting it on and taking it off with a trained healthcare professional. When performing the following instructions, refer to the overview image to locate the parts named in the text.

- A** elbow cuff
- B** elbow strap
- C** elbow clip
- D** motion damper
- E** thumb strap
- F** hook
- G** hand piece
- H** hand strap
- I** hand clip

Putting on the orthosis

- 1 Open the elbow clip (C) and hand clip (I) as shown in the overview image;
- 2 Raise your thumb and place the orthosis on the table on the outside of your arm;
- 3 Slide the thumb strap (E) over your thumb;
- 4 Place the elbow cuff (A) as low as possible around the upper arm and close the elbow strap (B) with the elbow clip (C). Tighten the strap so that it fits firmly around the arm;
- 5 Insert your thumb through the hand strap (H);
- 6 Push the hand clip over the hook (F);

- 7 Tension the hand strap by adjusting the hook-and-loop. Start with the strap under the thumb (7a), then the strap between the thumb and index finger (7b), and finish with the strap at the little finger end (7c).

Taking off the orthosis

- No need to undo the hook-and- loop to remove the orthosis;
- Open the handclip;
- Open the elbow-clip;
- Remove the orthosis from the arm and hand.

Fitting the fabrics

- 8a** Pass the two narrow straps of the hand strap (H) through the holes of the hand clip (I) as shown.
- 8b** Insert the wide strap of the hand strap (H) through the hand piece (G), making sure to close the hook-and-loop at the top.
- 8c** Snap the two press studs on the thumb strap (E) into the sockets in the hand piece (G).

Cleaning instructions

It is recommended to regularly clean the orthosis to maintain adequate levels of hygiene.

Textile parts

- Remove the textile parts from the orthosis;
- Fasten all hook-and-loop closures;
- The textile parts can be washed at temperatures up to 30°C, on delicate cycle or by hand;
- Airdry and do not expose to direct heat sources such as stoves or radiators.

Note: Do not tumble dry, iron, bleach, or wash with fabric softener.

Other parts

- Clean plastic and metal components with a damp cloth.

Maintenance

The product requires no maintenance if handled and cared for properly.

Disposal

In order to dispose of the orthosis, adhere to local regulations and instructions.

Report serious incidents

As imposed by local regulations, you are requested to immediately report any serious incident involving the use of this medical product to both manufacturer and relevant authorities.

Our contact details can be found on the back of this brochure.

Liability

STIL does not assume liability for improper use or unauthorized modifications of the product, nor for use outside the intended or recommended conditions, including against provided indications and contraindications.

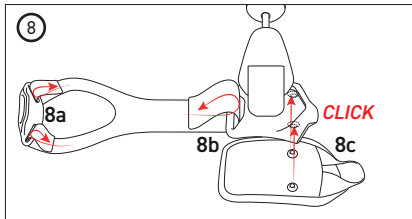
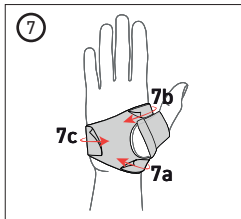
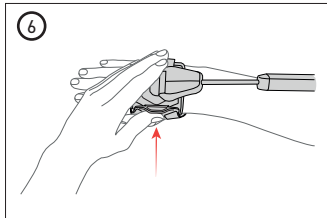
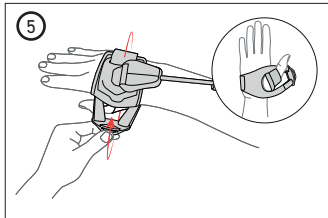
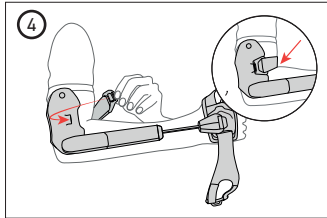
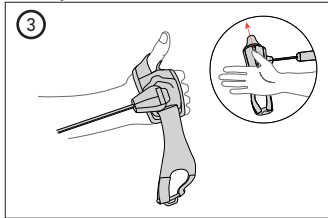
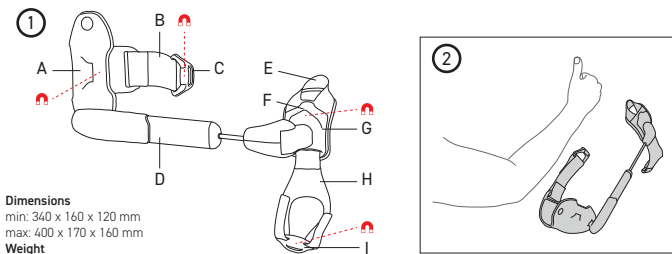
Repairs after warranty period may be available at a cost.

Materials

Textile parts contain:
 Nylon (polyamide), polyester, elastane, polyurethane, elastodiene.

Customer Support

In this manual, healthcare professional refers to the person who fitted and adjusted your orthosis. This may be a physical therapist, occupational therapist, certified prosthetist/orthotist (CPO), or another trained professional specialized in orthotic care. This person is your primary contact for any questions regarding fit, comfort, or use of the orthosis.

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